



Hearing Impaired

Help: patientcare@heartbug.com.au
Office hours 8am–5pm (Syd time) Mon-Fri

1. Checking that it's working

HeartBug will flash “hello” and “goodbye” when it connects and disconnects.

Hello = 3 green flashes. Goodbye = 3 red flashes.

These flashes tell you that HeartBug is working properly.





If you do not see these flashes, email us on patientcare@heartbug.com.au for help.

2. How to send

To use the landline, dial 1800 529 275. Press 1. Wait a few seconds. Press 1 again. Now you can send.

If you have a smartphone, it's much easier to use the app as there are no voice messages.

Press the red button on the HeartBug to start sending. There is a series of flashes during the sending process. Sending can take up to 4 minutes. When the flashes stop, it is finished.

			
Hello, it's me	Sending all the recordings (up to 4 mins)	Bye bye	No. of ECGs sent

3. How to record

When you press the red button, HeartBug will record for 40 sec.


The green light will flash during recording.

Nice to know!

The Memory

There's room for 8 recordings. You can check how many. Take off one lead. After 3 red “goodbye” flashes, count the green flashes:

Eg:  = 3 recordings

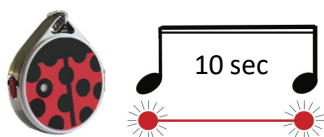

Long red flash = memory full.
Send now to clear memory.

Battery

If you see 3 long red flashes, then it's time to change your battery.



There's a spare one in your box. Here's how to do it:



Twist to remove Push out